

# Face Protection Shield

Clear, quality protection where it's needed most

- Enhanced personal protection against infectious droplets, bodily fluids, contaminants, etc.
- Fog-resistant
- Anti-glare, crystal clear lens
- Wearable with glasses/goggles and masks
- Secure elastic headband
- Comfortable: breathable and lightweight
- Full face protection: 13"W x 9"L
- Latex-Free



#40767

Case Qty: 60/ea

Ordering Info: (724) 337-5500

Or contact your sales representative



702 Prominence Drive  
New Kensington, PA 15068

Tel: 724-337-5500  
Fax: 724-337-0555

xodusmedical.com  
info@xodusmedical.com


ISO 13485 CERTIFIED  
FDA REGISTERED




LITBRIC003 R0 4/16/2020

# Xodus Medical Face Protection Shield Instructions for Use: Product #40767


## PLACING ON THE HEAD:

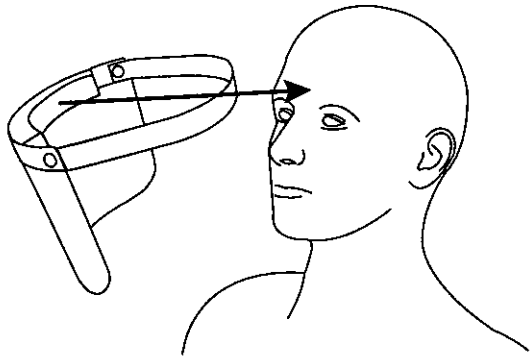

**1** Using one hand grasping the shield where the strap meets the shield, place the shield foam front against your forehead at a location above the eyebrows.

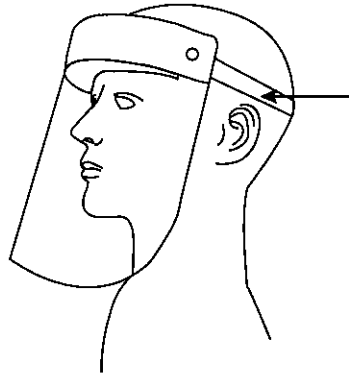


**2** While holding the foam front against your forehead, using your second hand, secure the elastic band around the back of your head so that the foam rests on your forehead.

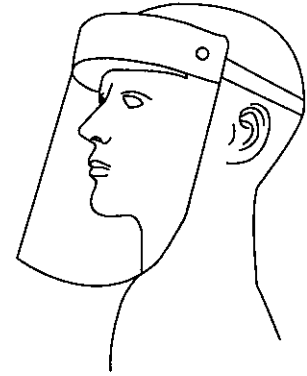

**3** Once shield is situated in a comfortable position, make sure the shield covers the front and sides of the head and that no areas are left uncovered.


**4** Adjust the Face Protection Shield as needed to maintain proper coverage.

## REMOVING FROM THE HEAD:

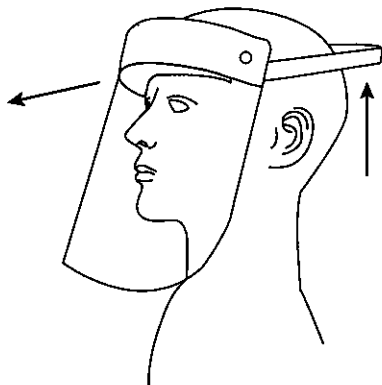

**1** Using one hand, grasp the front of the Face Protection Shield at the where the strap meets the shield.

**2** With your second hand, grasp the rear of the elastic band and lift the elastic band upward and forward above your head.

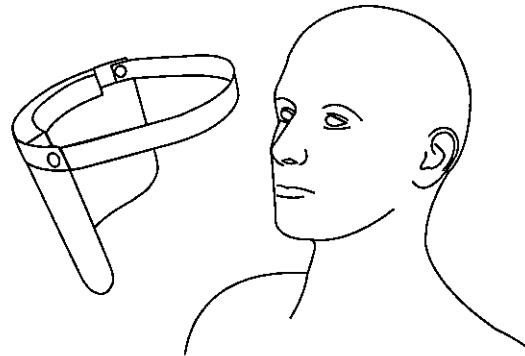

**3** Pull the shield away from your face.

**4** Dispose of the shield according to hospital protocol for the device.

## WHEN USING YOUR FACE MASK:

1. Wear the face protection shield in addition to other personal protective equipment
2. Replace if wet
3. Avoid exposure to high heat
4. Handle gently to prevent drops and scratches
5. Do not share with others
6. Shield may be disinfected during any single use; the shield may be disinfected based on hospital protocol for this type of device



702 Prominence Drive  
Pittsburgh, PA 15068 U.S.A.

Tel: 724.337.5500  
Fax: 724.337.0555



NON-STERILE



xodusmedical.com  
info@xodusmedical.com

FDA REGISTERED  
ISO 13485 CERTIFIED

LITIFUF5001 R0 4/17/2020